




ALPHA SERIES TREATMENT TABLES

MODELS 3010, 3020, 3030, 3013,
3100, 3200, 3300 & 3130

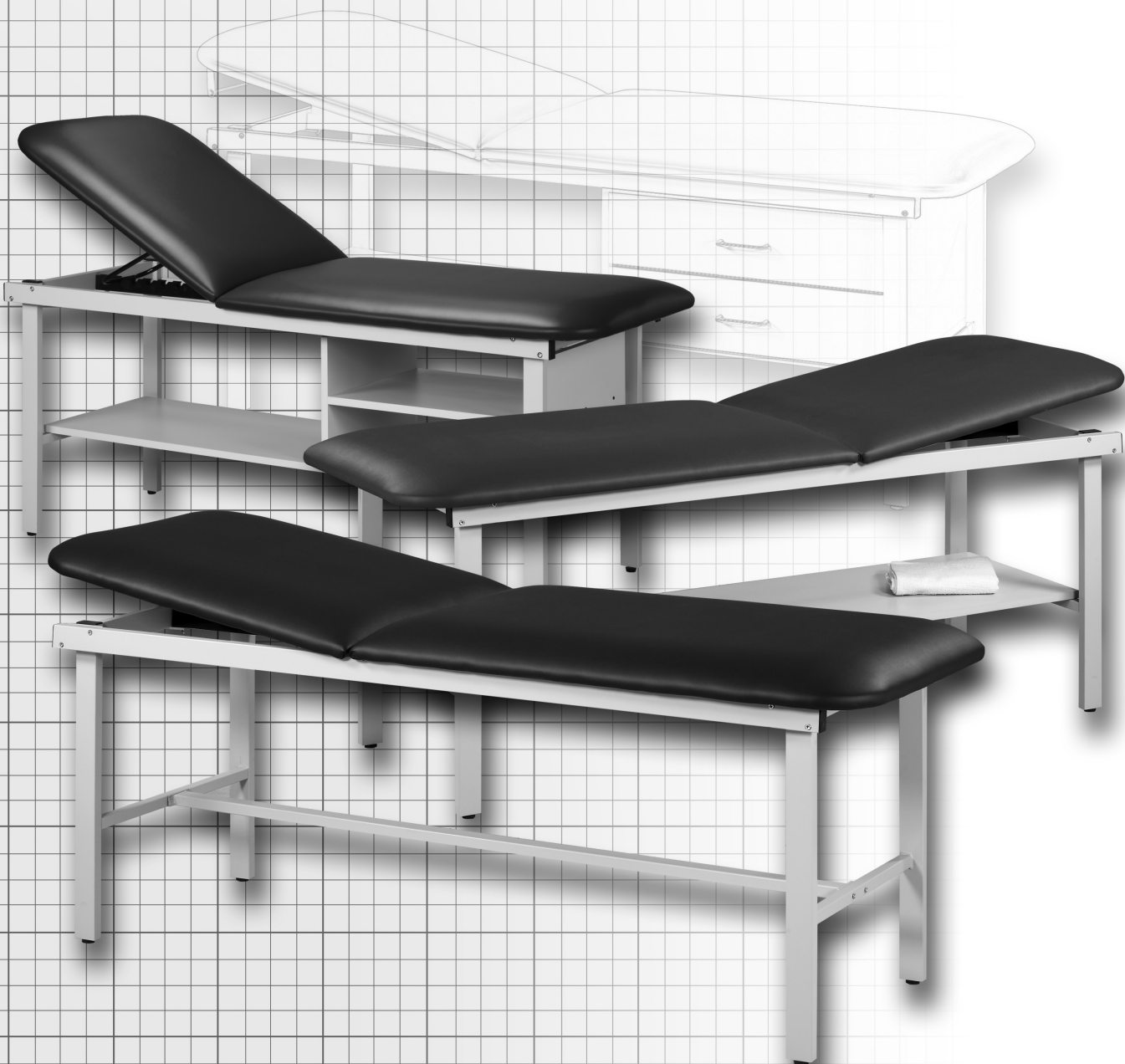

ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS FOR ALPHA SERIES TREATMENT TABLES

MODELS: 3010, 3020, 3030 & 3013 with ADJUSTABLE BACKREST/KD & Tables with Pre-Attached Backrest
For MODELS: 3100, 3200, 3300 & 3130 (with FLAT TOP and PRE-ATTACHED TOPS Skip STEPS 5 through 9)

MAXIMUM WEIGHT CAPACITY 450 LBS

Thank you for counting on Clinton to meet your medical equipment needs by purchasing this fine equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

INTENDED USE

Clinton treatment tables, models 3010, 3020, 3030 3013, 3100, 3200, 3300 & 3130 are specifically designed and engineered for the use by a single person at a time, who is attended to and observed by trained medical personnel. Never leave anyone on the table unattended. Person's weight and the combination of force exerted on the table during treatment is not to exceed a total of 450 lbs.

KD Quantity	Hardware Package	Pre-Attached Quantity
(4)	1/4-20 x 1 1/2" Hex Bolts	(4)
(4)	1/4-20 x 1" Hex Bolts	(4)
(2)	1/4-20 x 1" Truss Head Screws	(0)
(4)	1/4-20 x 2 5/8" Truss Head Screws	(0)
(2)	1/4-20 x 1" Truss Head Bolts	(0)
(8)	5/16" Washers	(4)
(6)	5/16" Locknuts	(0)
(4)	1/4" Locknuts	(4)
(4)	1 1/4" Black Truss Head Screws (3030 & 3013)	(4)

Tools Required

Wrench & (2) Allen Wrenches Provided
 Phillips Screwdriver and Adjustable Wrench (not provided)

1 Open the frame box, remove all packaging and position the table frame upside down on a soft surface that won't mar the top. Position the legs up as shown below.




2 Use the hardware and tools provided to secure the legs into a locked position using the 1/4-20 x 1 1/2" hex bolts and 1/4" locknuts. **DO NOT** to tighten at this time.



3 Attach the spanner between the two legs using the 1/4-20 x 1" hex bolts and wrench provided and **completely tighten all 12 frame connecting hardware points from step 2**



4 Spin the 4 adjustable feet into the legs. With assistance, invert the table into an upright position on all four legs, using caution when inverting the end with the adjustable headrest.

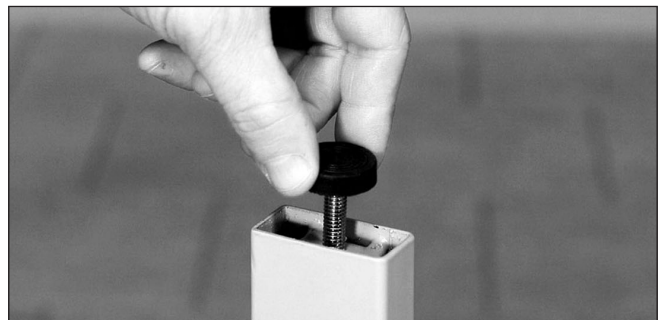

FOR MODELS WITH ADJUSTABLE BACKREST

5


Place the black backrest channel on top of the frame tabs with the channel cutouts close to the head end. Secure using (2) $\frac{1}{4}$ -20 x 1" truss head screw and $\frac{5}{16}$ " locknut at each end.



FOR MODELS WITH ADJUSTABLE BACKREST

6


Loosely fasten the adjustable backrest hinge to the frame's cross member using (4) $\frac{1}{4}$ -20 x $1\frac{1}{2}$ " hex bolts, two $\frac{5}{16}$ " washers (one each side) and $\frac{5}{16}$ " locknuts. *Note: Hinge flap goes on the black channel side.*

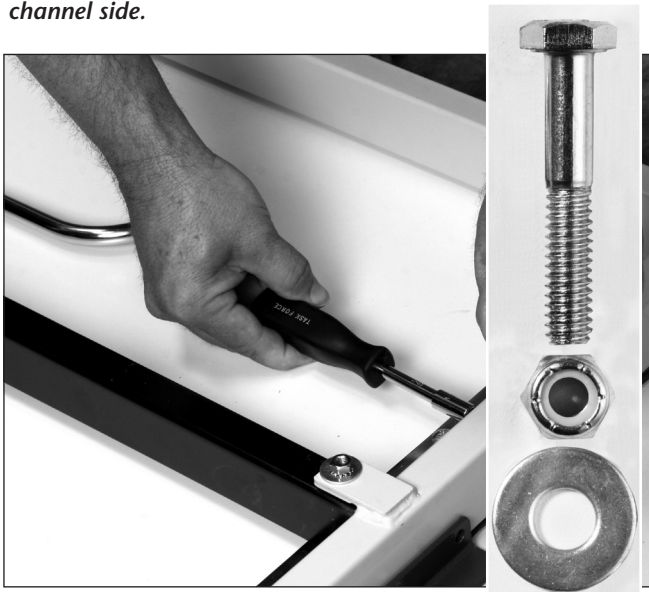

FOR MODELS WITH ADJUSTABLE BACKREST

7


Mount the foot end of the top by placing the (4) $\frac{1}{4}$ -20 x $2\frac{5}{8}$ " truss head screws through the frame sides and the (2) $\frac{1}{4}$ -20 x 1" truss head screws and $\frac{5}{16}$ " washers at the end mounting points and tightly securing the top. **Fully tighten hinge hardware from Step 6.**

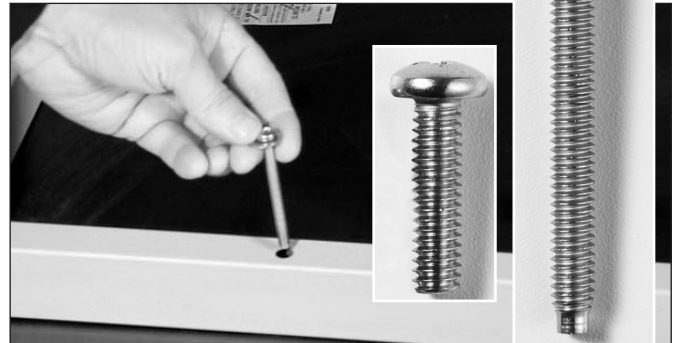

FOR MODELS WITH ADJUSTABLE BACKREST

8

Secure all screws in the frame

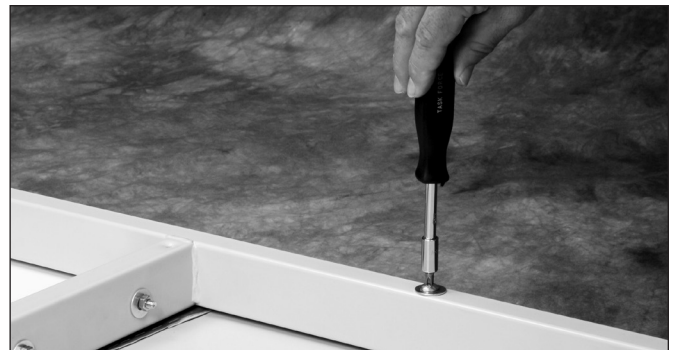

FOR MODELS WITH ADJUSTABLE BACKREST

9

Secure the middle brace to the top



FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL SCREWS, BOLTS AND NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE TABLE HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.

PROCEED TO STEPS 10, 11 & 12 ON BACK COVER FOR MODELS: 3020, 3030, 3013, 3200, 3300 & 3130

10

3020/3200 SHELF ASSEMBLY

Invert table to upright position and apply 4 adhesive backed hook and loop strips to top edge of H-braces. Place the full-length shelf on the spanner with the loop strip underside over the hook strips on the braces.



11

3030/3300 TIER SHELF ASSEMBLY

With table in the upright position, place the full-length shelf into position under the table (see step 10). Using assistance, place tier shelf on the full shelf, aligning it with the end of the shelf that has the pre-drilled holes. (See photo below)



12

3013/3130 DRAWER ASSEMBLY

With table in the upright position, place the full-length shelf into position under the table (see step 10). Remove the drawers and place the drawer casing into position aligning it with the end of the shelf that has the pre-drilled holes. Fasten unit by screwing the supplied 1 1/4" black truss head screws through shelf and into pre-inserted, threaded inserts. (See photo below).



CARE & CLEANING

Clinton products are built to be durable. With proper care, your Clinton equipment will provide years of superior looks and performance. Clean table with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Before using any cleaning agents, try them on a small inconspicuous area to insure that they will not harm the any surface. Do not use alcohol based cleaning agents.**

Periodically check that all fasteners are secure and operating properly.



800-441-9131 • 717-848-3519
Toll-Free Fax 866-522-1233
Fax 717-843-5871
www.clinton-ind.com