Comfort Cool® D-Ring Wrist Orthosis (Regular and Long)

Instructions for Care

Hand wash in cool water with mild detergent and air dry.

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Regular		Long		Wrist Circumference	
Left	Right	Left	Right		
NC52960	NC52961	NC52970	NC52971	X-Small	Up to 5¾" (Up to 15cm)
NC52962	NC52963	NC52972	NC52973	Small	5¾" to 6½" (15 to 17cm)
NC52964	NC52965	NC52974	NC52975	Medium	6½" to 7½" (17 to 19cm)
NC52966	NC52967	NC52976	NC52977	Large	7½" to 8½" (19 to 22cm)
NC52968	NC52969	NC52978	NC52979	X-Large	8½" and over (23cm and over)

Long measures 2" (5.1 cm) longer than **Regular**.

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ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 XSR3, Ireland

ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom

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D-Ring Wrist Orthosis

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D-Ring-Handgelenkorthese

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Ideal for carpal tunnel syndrome, arthritis, tendinitis or other painful wrist conditions.

Includes a removable, adjustable metal stay to provide support along the wrist and palm.

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Comfort Cool® D-Ring Wrist Orthosis (Regular and Long)

Intended Purpose

Versatile orthosis designed with removeable metal stay to provide wrist support, stabilization, and adjustable positioning to promote healing and pain relief.

Indications

Ideal for carpal tunnel syndrome, arthritis, tendinitis or other painful wrist conditions.

Contraindications

Loss of circulation and skin irritation or allergy to polyester, nylon or neoprene.

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Instructions for Use

1. Unfasten the straps and slide the hand into the splint. Do not insert the hand between the layers of the stockinette. Be sure that the metal stay is placed on the palm side of the hand.
2. Slide the splint as far back on the arm as possible. The palmar end of the splint should lie below the palmar crease.
3. Refasten the straps so it feels snug but not so tight to interfere with circulation. Trim the straps or neoprene areas for a more custom fit, but do not cut sewn seams.
4. The metal stay can be re-bent to fit the hand and wrist for a more precise fit. Slide the metal stay to the finger end of the pocket.

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