Comfort Cool® Thumb CMC Abduction Orthosis Instructions

Instructions for care Hand wash mild detergent in cool water and air dry.

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
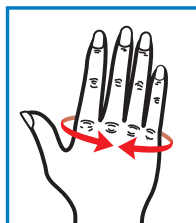
Plus (+) sizes are a half size larger in the hand and a full size larger in thumb circumference.

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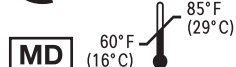MD

60°F
(16°C)

Warning Follow instructions for microwave heating and molding of the orthosis. Direct contact with heated thermoplastic insert may result in burning the skin.

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EC REP ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 X5R3, Ireland

UK REP ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom

CH REP Allenspach Medical AG
Dornacherstrasse 5
4710 Balsthal, Switzerland

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Helps to treat thumb contractures and conditions, especially where the thumb is held abducted against the palm.

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Indications: Thumb deformity and contractures due to neurological conditions, median nerve injuries, arthritis, medial subluxation of the thumb CMC, joint instability, sprain, tendinitis, or repetitive motion injuries.

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Contraindications: Loss of circulation and skin irritation or allergy to polyester, nylon, neoprene, or spandex.

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- Slide thermoplastic insert **fully** inside strap's pocket (1A). Trim insert if needed. Carefully heat orthosis in microwave in **20 second intervals** until insert is soft (1B).

Microwave CAUTION: Insert piece will be very hot and sticky and cannot be removed after heating. Any exposure of thermoplastic insert may result in burning the skin.

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Optional heating method: Immerse orthosis strap containing plastic insert into 160°F (71°C) water until soft. Proceed to instruction number 2 for fitting.

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- Secure orthosis onto the hand with thumb sliding through the thumb hole and secure the hook and loop. Place thumb in comfortable abduction/extension. Align warm, pliable strap over the CMC joint. Pull the strap firmly, following palm's curved hook fastener to apply supportive tension along the first metacarpal (2).

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- Bring strap through web space without stretching it. Fasten end to small hook piece (3). Hold thumb in place until insert hardens. Insert can be reheated and reshaped for progressive positioning if needed.

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