## Comfort Cool® Thumb CMC Restriction Splint Instructions

### Instructions for care

Hand wash mild detergent in cool water and air dry.

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
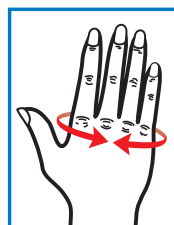
Plus (+) sizes are a half size larger in the hand and a full size larger in thumb circumference.

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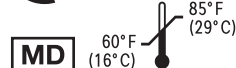EC REP ICON (LR) Limited  
South County Business Park  
Leopardstown, Dublin 18  
D18 XSR3, Ireland

UK REP ICON (LR) Limited  
500 South Oak Way  
Green Park, Reading  
RG2 6AD, United Kingdom

CH REP Allenspach Medical AG  
Dornacherstrasse 5  
4710 Balsthal, Switzerland

**Warning** If numbness or discomfort occurs, discontinue use and seek medical advice.

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North Coast Medical, Inc.  
www.ncmedical.com

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Daumen-CMC-Restriktionsschiene

Tumbasortos

Helps relieve thumb and CMC joint discomfort associated with arthritis, joint instability, sprain, tendinitis or overuse.

Unique strap wraps around the base of the thumb CMC joint, providing additional support.

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## Comfort Cool® Thumb CMC Restriction Splint Instructions

### Intended Purpose

Provides support to the thumb and CMC joint for pain relief, stiffness, and inflammation.

### Indications

Osteoarthritis and medial subluxation of the thumb CMC joint. Relieve thumb discomfort associated with CMC joint arthritis, instability, sprain, tendinitis, or repetitive motion injuries.

### Contraindications

Loss of circulation and skin irritation or allergy to polyester, nylon, neoprene, or spandex.

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© 2024 North Coast Medical, Inc.