Lightweight, comfortable support.

# Ulnar Booster™

## Instructions for Use / Proper Fit • Instrucciones de uso / Ajuste correcto • Mode d'emploi / Ajustement • Gebrauchsanweisung / Richtige Bruksanvisning/korrekt tillpassning

1. Check that the wrist strap is through the slit in the splint. ●  
Comprobar que la correa para la muñeca esté insertada a través de la ranura de la férula. ●  
Vérifiez que la sangle du poignet est enfilée dans la fente de l'attelle. ●  
Überprüfen, dass der Handgelenkriemen durch den Schlitz in der Schiene gezogen ist. ●  
Kontrollera att handledsremmen är igenom slitsen i skenan. ●

2. Slide the hand under the wrist strap and place the thumb in the hole. ●  
Deslizar la mano por debajo de la correa para la muñeca y meter el dedo por el hueco. ●  
Enfilez la main sous la sangle et placez le pouce dans le trou. ●  
Die Hand unter den Handgelenkriemen schieben und den Daumen durch die Öffnung stecken. ●  
För in handen under handledsremmen och placera tummen i hålet. ●

3. Place the wrist strap over the raised bony area (ulnar styloid) [II]. ●  
Colocar la correa para la muñeca por encima de la zona ósea abultada (apófisis estiloides del cúbito) [I]. ●  
Placez la sangle de poignet sur la partie osseuse en saillie (styloïde cubitale) [I]. ●  
Den Handgelenkriemen über dem vorstehenden knöchernen Bereich (Griffelfortsatz der Elle, Processus styloideus ulnae) [I] platzieren. ●  
Placera handledsremmen över det upphöjda beniga området (ulnara styloiden). ●

4. Feel next to the wrist crease on the ulnar palm area for the pea-size pisiform bone [III]. ●  
Palpar junto al surco de la muñeca en la zona cubital de la palma de la mano para encontrar el hueso pisiforme (en forma de guisante). ●  
Tâtez à proximité du pli du poignet au niveau de la paume, côté cubitus, l'os pisiforme [III] de la grosseur d'un pois. ●  
Neben der Handgelenkfalte am ellenseitigen Handflächenbereich das Erbsenbein [III] ertasten. ●  
Känn efter det ärtstora os pisiforme intill handledsfåran i handflatan på den ulnara sidan. ●

5. Place the foam pad inside the splint [III], lined up with the pisiform bone area. ●  
Colocar la almohadilla de espuma [III] dentro de la férula alineándola con la zona del hueso pisiforme. ●  
Placez l'insert en mousse [III] à l'intérieur de l'attelle, dans l'alignement de l'os pisiforme. ●  
Das Schaumstoffpolster [III] auf der Innenseite der Schiene in Ausrichtung mit dem Erbsenbeinbereich platzieren. ●  
Placera skumplastdynan innanför skenan, över os pisiforme-området. ●

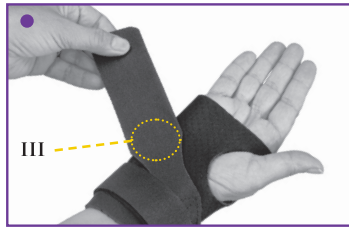
6. Bring the splint edges together along the back of the hand and fasten. ●  
Juntar los bordes de la férula a lo largo del dorso de la mano y ajustarlos. ●  
Rassemblez les bords de l'attelle le long du dos de la main et fixez-les en place. ●  
Die Schienenkanten an der Handrückseite entlang zusammenbringen und befestigen. ●  
För upp skenans ändrar över handryggen och sätt fast dem. ●

7. Wrap the wrist strap around and fasten. The strap should provide compression over the ulnar styloid. ●  
Colocar la correa para la muñeca alrededor de la férula y ajustarla. La correa debe comprimir la apófisis estiloides del cúbito. ●  
Enroulez la sangle de poignet autour et attachez-la. La sangle doit comprimer la styloïde cubitale. ●  
Den Handgelenkriemen um das Handgelenk wickeln und befestigen. Der Riemen sollte oberhalb des Griffelfortsatzes der Elle Kompressionsdruck ausüben. ●  
Linda handledsremmen runt handleden och sätt fast den. Remmen ska ge kompression över den ulnara styloiden. ●

Instructions for Use / Proper Fit • Instrucciones de uso / Ajuste correcto • Mode d'emploi / Ajustement • Gebrauchsanweisung / Richtige  
Bruksanvisning/korrekt tillpassning


8. Bring the diagonal strap over the foam pad area and fasten on the back of the hand. This strap should provide compression on the foam pad, to "boost" the ulnar wrist (carpal) bones. ●

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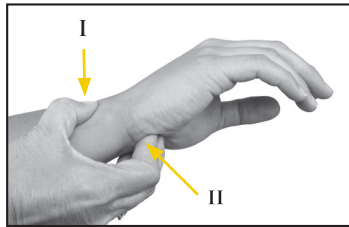9. Once fitted correctly, remove and permanently stick the foam pad to the proper spot inside the splint. The pad should not be moved again. ●

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10. The straps and splint edges can be trimmed with scissors, if needed. ●

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When worn properly, the Ulnar Booster Wrist Splint should feel like a counterforce on the ulnar side of the wrist.

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**Instructions for Care:** Hand wash in cool water with mild detergent and air dry.

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
## Sizing
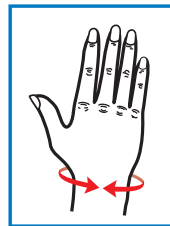
| Left      | Right     |         | Wrist Circumference |               |
|-----------|-----------|---------|---------------------|---------------|
| NC68030-1 | NC68030-2 | X-Small | 4" to 5"            | (10 to 13 cm) |
| NC68030-3 | NC68030-4 | Small   | 5" to 6¼"           | (13 to 16 cm) |
| NC68030-5 | NC68030-6 | Medium  | 6¼" to 7¼"          | (16 to 18 cm) |
| NC68030-7 | NC68030-8 | Large   | 7¼" to 8½"          | (18 to 22 cm) |

To be used under the guidance of a qualified medical professional.

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**WARNING:** If any numbness or discomfort occurs while wearing the splint, discontinue use and consult your health care professional.

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ICON (LR) Limited  
South County Business Park  
Leopardstown, Dublin 18  
D18 X5R3, Ireland

ICON (LR) Limited  
500 South Oak Way  
Green Park, Reading  
RG2 6AD, United Kingdom

Allenspach Medical AG  
Dornacherstrasse 5  
4710 Balsthal, Switzerland

North Coast Medical, Inc.  
www.ncmedical.com

680301FU-49-V3.0 2024  
© 2024 North Coast Medical, Inc.

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Morgan Hill, CA 95037 – U.S.A.