

### Intended Purposes

Non-sterile device used to assist in pulling on sock or stockings, with use of a single or double cord.

### Indications

Persons with difficulty bending over, dizziness, or vertigo.

### Contraindications

Limited hand strength. Allergy to polypropylene.

### Instructions for Use



Gather entire sock over end of sock aid.



Hold strap and drop sock aid onto floor.



Slide foot into sock aid opening.



Pull upward on strap to bring sock over heel and up leg.

### Warning:

Keep out of reach of children. Plastic bag and small parts may pose a choking hazard.

### Instructions for Care

Hand wash in cool water and mild soap. Air dry.

*Note: Cord handle can be cut and securely knotted to adjust length.*

