Comfort Cool® Gladiator™ Short Wrist Orthosis

LEFT

RIGHT

NC86640	NC86641	S	6" to 7" (15 to 18 cm)
NC86642	NC86643	M	7" to 8" (18 to 20 cm)
NC86644	NC86645	L	8" to 9" (20 to 23 cm)
NC86646	NC86647	XL	9" to 10" (23 to 25 cm)

 North Coast Medical, Inc.
135 E Main Ave, Suite 110
Morgan Hill, CA 95037 - U.S.A.

 **EC REP** ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 X5R3, Ireland

 **UK REP** ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom

 **CH REP** Allenspach Medical AG
Dornacherstrasse 5
4710 Balsthal, Switzerland

60°F (16°C) 85°F (29°C)

 North Coast Medical, Inc.
www.ncmedical.com

© 2024 North Coast Medical, Inc.

86640IFU-159-V1.0

GLADIATOR™

Short Wrist Orthosis

Férula corta para muñeca
Orthèse de poignet courte
Kurze Handgelenkorthese
Kort handledsortos

INSTRUCTIONS FOR USE

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MODE D'EMPLOI

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BRUKSANVISNING

GLADIATOR™

Short Wrist Orthosis

Intended Purpose

Versatile orthosis designed with removeable metal inserts to provide firm wrist support, stabilization, and adjustable positioning to promote healing and pain relief. Suitable for adults ranging from small to X-large sizes and in various clinical settings.

Indications

Wrist sprains, triangular fibrocartilage complex (TFCC) injuries, postsurgical distal radius fractures, wrist instability, and ulnar drift.

Contraindications

Loss of circulation and skin irritation or allergy to synthetic fabrics, synthetic rubbers, or cotton.

Instructions for Use / Proper Fit

- 1 Loosen all straps and slide hand into the orthotic with the support stay on the palm.
- 2 Slide the orthotic backward on the forearm, below the palmar crease. If necessary, bend the three metal stays for a custom contouring fit before strapping.
- 3 Begin strapping with the forearm strap closest to the bicep. Route strap over the forearm into the D-ring loop. Secure straps snugly. Refit as necessary to avoid interference with circulation. Continue counterforce strapping toward the wrist.

Refer to strapping diagram for guidance.

Instructions for Care

Remove all metal stays. Hand wash in warm water with mild soap and air dry. Machine washing and drying may damage the support.

Warning

Discontinue use or adjust fit if patient shows signs of irritation, impaired circulation, increased pain or discomfort, or allergic reactions such as redness, itching, tingling, rash, or color changes in the affected area.

To be used under the guidance of a qualified medical professional.

Guaranteed against defects in material or workmanship.
Replacement will not be made for products misused or abused.

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