



3-Section Hi-Lo Table, Manual Hydraulic



3-Section Hi-Lo Table, Manual Hydraulic

CAUTION: THE MOBILE OPTION IS ONLY FOR MOVING THE UNWEIGHTED TABLE SHORT DISTANCES ON A SMOOTH FLOOR. AVOID UNEVEN SURFACES SUCH AS TRANSITIONS BETWEEN ROOMS AND ELEVATORS.

Maintenance

- Hydraulic Pump does not require maintenance. Should any problems occur with this unit, please contact Customer Service at 1-800-821-9319.

How to Care for your Upholstery Fabric

Here are some steps to protect the table's upholstery and ensure a long life for the product.

- Do not store the table in extremely cold temperatures — excessive cold will cause the upholstery to crack. Keeping the table in an unheated area during the winter is not recommended.
- Do not store the table in extremely warm temperatures since excessive heat will cause the upholstery to stretch and pucker.
- Do not place the table in direct sunlight. This will prematurely fade and dry out the upholstery and will void the upholstery warranty.
- Clean the table's upholstery after each day of use. We recommend cleaning the upholstery with a mild detergent. A common practice is to use a spray bottle filled with a mixture of anti-bacterial dish soap and water on the upholstery, followed by a wipe down with a clean towel. Rubbing alcohol will dry out the upholstery and cause it to crack.
- Protect the table from general abrasions, cuts and punctures.

Warranty

Table comes with a lifetime warranty on steel structures and welds, and a three-year warranty on the hydraulic pump, movable parts, hardware, and accessories. Table upholstery has a three-year warranty for manufacturer-related defects but does not cover damage from improper cleaning, cuts, abrasions, scratches, or normal wear and tear. All warranties exclude damage caused to the table by improper set-up, accident, abuse, or use other than the table's intended purpose.

No registration is necessary. An original sales receipt, order number, or other proof of purchase is required in order to redeem the warranty.



REG. NO. CA 47596



North Coast Medical, Inc.
135 E Main Ave, Suite 110
Morgan Hill, CA 95037 – U.S.A.



North Coast Medical, Inc.
www.ncmedical.com

MADE IN CHINA

Instruction Manual



Intended Use: Adjustable articulated table for lifting/lowering and positioning person(s) of various sizes.

Weight Limit

(including patient and accessories):

500 lbs / 225 kgs Lifting Capacity

NC94700

3-Section Hi-Lo Table, Manual Hydraulic

Warnings

- Improperly setup, operated or poorly maintained equipment may cause injury.
- Equipment is not intended for use with children.
- To avoid danger of suffocation, keep plastic wrap and packaging components away from babies and children when unpacking the table.
- Persons who suffer from existing allergies, asthma, or other respiratory conditions may adversely react to the high density foam.
- Do not use equipment if there are signs of damage.
- The upholstery, polyvinylchloride, may contain phthalates, chemicals known to the state of California to cause cancer or reproductivity toxicity.
- Use caution when limbs/fingers are near moving parts as injury may occur.
- Do not stand on table as injury may occur.
- Do not use as a step ladder as injury may occur.
- Use of chemicals on or around the hydraulic pump may cause an adverse reaction and result in injury.
- Use caution when weight is unevenly distributed as table may tip over and result in injury.
- Do not use on a sloped surface as it may cause table to tip over and result in injury.
- Use caution when weight is near lift capacity (500 lbs / 225 kgs).
- Use caution when table is at maximum extended height.

Complies with U.S. CPSC requirements for upholstered furniture flammability. Complies with Technical Bulletin (TB) 117-2013, "Requirements, Test Procedure and Apparatus for Testing the Smolder Resistance of Materials Used in Upholstered Furniture."

Safety Instructions

Read the following very carefully and ensure that all staff members who operate the table are fully briefed on safety compliance.

- Use both hands to adjust the angle or the height of any section of the table.
- Do not adjust foot or head sections while in use.
- Check that the table is leveled before use.
- Do not put excessive weight or pressure on the head or foot section.
- Check the equipment frame, wheels, hydraulic case, padding, and upholstery when unpacking the table. Do not use it if there are signs of damage.
- When the table is in operation, do not open the hydraulic pump, which is located below the table.

Instructions For Use

Adjusting the Height of the Table

- The table comes with foot levers to adjust the height.
- Push the UP lever to raise table.
- Push the DOWN lever to lower table.
- Confirm table is leveled before use.
- Keep area beneath the table and cushions clear to avoid injury or damage to table.



Adjusting the Head and Foot Section

- Adjust the angle of unweighted head or foot sections by pulling up on hand lever to raise or lower sections.



Using the Wheels to Move the Table

The table is equipped with positioning levers on all four corners to engage/disengage the wheels.

- **To move table:** place your foot on the positioning lever and press *downward* to set the wheels onto the floor. The table can now be moved.
- **To lower table to floor:** lift positioning lever *up* with your foot to retract the wheels. The table is now stationary.

