



# 3-Section Hi-Lo Table Electric

## UNPACKING YOUR TABLE

Thank you for your purchase! Even though your table has been packed and shipped with care, we recommend keeping the original packaging until final inspection has taken place.

### For your safety, two people are needed to remove the 3-Section Hi-Lo Table from the carton.


- Carefully remove each side of the shipping box.
- Push down on the Caster Control Bar under the leg section of the table to activate the casters. See photo below.
- Lift one end of the table at a time from the lower metal frame (not from the cushions), and lower to the floor. Be careful not to damage any wiring or power cords that might be hanging down under the table while moving the table off the wooden base.
- Keep the Spare Parts bag in case you need to replace foot pads, screws or stabilize the table with the Adjustable Foot Pad.

Inspect the table for damage before use. Check the frame, wheels and upholstery when unpacking the table. Do not remove from packaging if there are signs of damage.

If any damage is found at the time of delivery, take photos, save all packaging and contact North Coast Medical, Inc. Customer Service at 1-800-821-9319.

### Before Use:

- Remove all packaging materials, including straps that secure the cushions to the metal base.
- To avoid danger of suffocation, keep plastic wrap and packaging components away from babies and children when unpacking the table.
- Place the table where it is needed.
- Make sure the 3-Section Hi-Lo Table has clearance to move up and down freely.
- Connect electrical plug into a power socket.

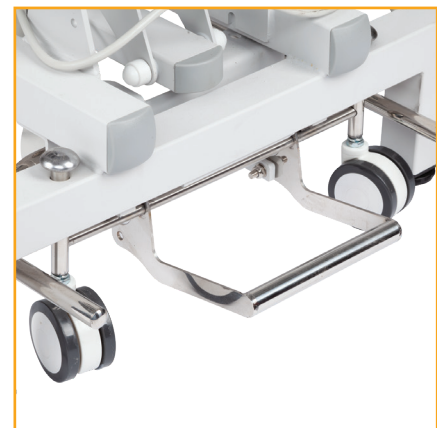

*Caster Control Bar*