Norco® Finger Extension Tube

Intended Purposes Assist PIP and DIP joints into increasing extension and correcting flexion contractures. Circumferential pressure helps in the restriction of fluid to aid in edema reduction.

Indications PIP and DIP joint flexion contractures.

Contra Indications: Skin irritation, allergy to nylon or neoprene, or fluctuating edema.

Instructions for Use / Proper Fit

To size, measure PIP joint circumference. Slide onto affected finger with the covered seam under the PIP/DIP joint. Tip of the finger should be visible. If needed, trim either or both ends for best fit. Splint should fit comfortably. A firm fit is not required for effective results. If too tight, size up. Avoid direct contact with open wounds. If skin becomes irritated, add a thin liner such as stockinette over affected skin area.

Note: Not for flexion contracture greater than 80°.

Warning: If fingertips change color, skin becomes irritated, or if pain, stiffness or numbness increases, discontinue use and seek advice from a healthcare professional.

Instructions for Care Spot clean with a damp sponge and air dry.

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North Coast Medical, Inc.
Morgan Hill, CA 95037 - U.S.A.
ICON (LR) Limited
Allenspach Medical AG