



Intended Purposes

A non-sterile vibration device used to loosen skin adhesions, reduce the buildup of scar tissue, desensitization, muscle stimulation, oral stimulation, and sensory re-education.

Indications Reduce skin adhesions and scar build up resulting from trauma or surgery. Reduce sensitivity, stimulate muscles (including oral tone), and loss of sensation.

Contraindications Allergy to acrylonitrile butadiene styrene (ABS).

Instructions for Use

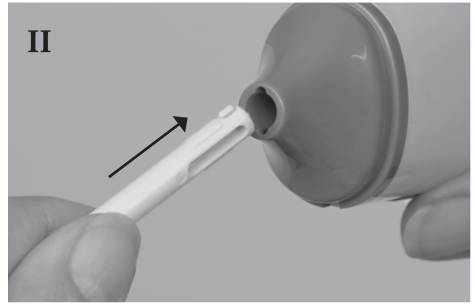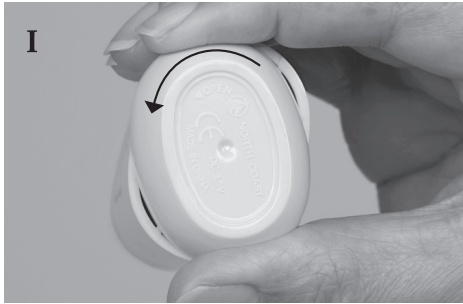

Battery Installation: Remove plastic from included battery. To open battery compartment (located on bottom of device) twist battery cap to the left and pull away from unit. (I) To Insert battery, slide the + terminal inside the unit so it snaps into place. Position cap slightly to the left, push and rotate cap into place. Twist to the right to lock.

Attachment Installation: Select desired massage attachment. Align tabs and slide completely into housing.(II)

To remove attachment, grip the stem and pull out of the housing.

For power issues: check for proper battery insertion and +/- orientation; replace battery as needed.

For attachment or noise concerns: align and insert attachments fully. Discontinue use if device generates unusual noises and inspect for damage.

Caution: Do not use on the throat or around the eye area. Keep away from humid areas and do not immerse in water. Remove battery when not in use for long periods of time.

Warning: Keep out of reach of children. Plastic bag and small parts may pose a choking hazard.


Instructions for Care

Clean between uses. Wipe vibrator unit and massage head with a mild disinfectant. Air dry.

Limited Warranty: 90 Days

Specifications

Power Source:	DC 1.5 V (1 x "C" cell)
C.P.S.:	92 CYCLES PER SECOND
Amplitude:	.06mm
Size:	1-5/8" X 1-1/4" X 5-3/8" (4.1 x 3.2 x 13.7cm)
Weight:	5.1 oz. (145g) with battery

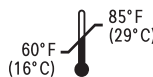

North Coast Medical, Inc.
135 E Main Ave, Suite 110
Morgan Hill, CA 95037 - U.S.A.



ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 X5R3, Ireland



ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom



Allenspach Medical AG
Dornacherstrasse 5
4710 Balsthal, Switzerland

