


ADJUSTING PVX

MCP & STABLE THUMB SPLINTS

Without tools, bend the PVX closer or farther from palm to provide support under joint and block collapsing. PVX may need to be twisted for more even pressure. If PVX cannot be adjusted by the therapist, return to SilverRing™ Splint to assist with adjustments.

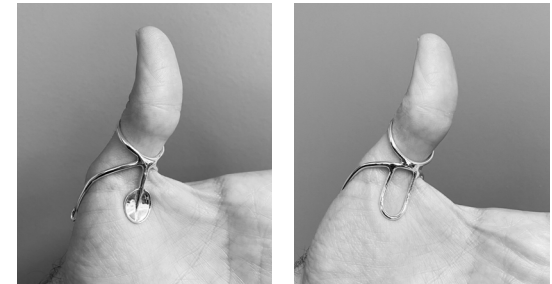

The Spoon PVX gives more directional support and the U PVX avoids that direct pressure.

Care Instructions

SilverRing™ Splints are made of sterling silver, a silver-copper alloy which is 92.5% pure silver and 7.5% copper. Sterling silver contains NO nickel. They can be cleaned as you would with any sterling silver products. Tarnish can occur with reactions to body chemistry, medications, hand lotions, zinc oxide and exposure to chemicals like hot tubs or pools. Foods like eggs, onions and some shellfish can also cause a reaction. 14k Gold can be an option for someone who will be in constant contact with such chemicals on a regular basis.

Repairs/Reshapes/Returns

Bent or misshapen splints can be reshaped and price will be based on labor. Broken PVX's can be repaired. Cracked or broken splints should be replaced. Standard splints may be returned while still new. If adjusting does not work, place the splint(s) back in a box for return. Write changes needed directly on the back of the invoice or packing slip. Return authorization is not required but please include phone and email.

Ship everything to:
1140 E Market St. Charlottesville VA 22902

Our Guarantee

The Silver Ring Splint Company warrants its products against defects in materials and workmanship for one year. Should any product fail to perform for either of the above reasons, the Silver Ring Splint Company will repair or replace the product at no charge. **Repairs by anyone other than the Silver Ring Splint Company will void our warranty.**




Thank you for using our products.
For additional information please visit our website
at www.SilverRingSplint.com



Phone: (434) 971-4052
info@silverringsplint.com

Adjustments, Repairs Returns & Reshapes

GUIDE FOR THE FOLLOWING SPLINTS

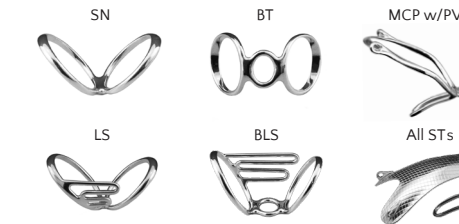

*Swan Neck (SN), Boutonniere (BT), MCP with PVX (MCP w/PVX),
Lateral Support (LS), Boutonniere Lateral Support (BLS)
and all Stable Thumb splints*

Silver Ring Splint
C O M P A N Y

SilverRing™ Splints

To ensure a custom fit, splints may need to be adjusted. Making small adjustments by hand will not affect the integrity of the splint or the ability to return for exchange or refund. **Do NOT use tools on the splints** as it will void our warranty and make the splint non-returnable. Please use the enclosed instructions when adjusting your splint. Follow the return instructions in this insert which will expedite the exchange or refund.

MAKING THE SPLINT FIT

LARGER

The splint size can be increased by about a size by decreasing the angle between the rings. Place the splint on a flat surface and using even pressure, bend rings closer together. Make small adjustments and check the fit between each adjustment.



Squeezing the rings together loosens the fit of the splint. This adjustment can allow a finger to reach a neutral position if the splint is holding the finger in slightly bent position.

MAKING THE SPLINT FIT

SMALLER

The splint size can be decreased by about a size by increasing the angle between the rings. Place the splint on a flat surface and using even pressure, bend rings further apart. Make small adjustments and check the fit between each adjustment.



Spreading the rings apart tightens the fit of the splint. This adjustment can hold a finger in a slightly bent position and may help avoid locking or clicking in the joint.

ADJUSTING THE

BRACELET

MCP & STABLE THUMB SPLINTS

Use a twisty tie or string to temporarily adjust bracelet. After trialing length, a local jeweler can also adjust bracelet which **does NOT void warranty**. Links cannot be added back after adjustment. Bracelet can also be permanently adjusted at no cost by SilverRing™ Splint.



Bracelet can be worn in multiple ways and should be trialed before final adjustment. If patient wishes to crisscross bracelet a longer length can be requested at no cost.